



Mindray D30 Defibrillator Training resources

VIDEO

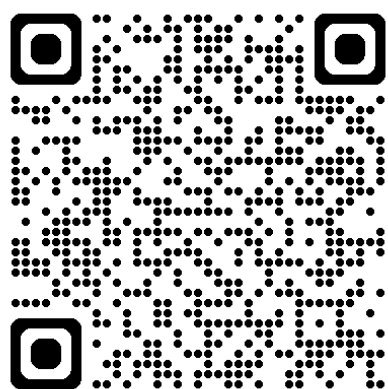

QR CODE
SCAN ME

Advanced USER GUIDE

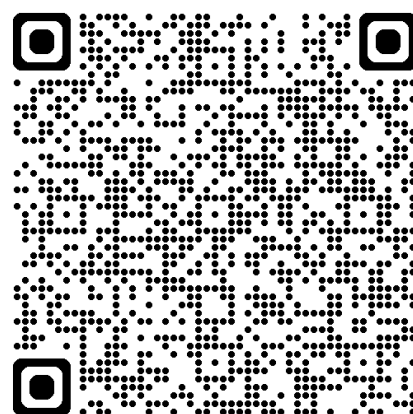

QR CODE
SCAN ME

Find more resources on the
Medical Device Training page



SCAN HERE



Mindray D30 Defibrillator User check

The D30 defibrillator will automatically perform a automated self-test each morning when attached to test plug.




- If the device passes the self-test a green spanner will be visible on the screen.



- A red flashing spanner, accompanied by an audible beep denotes the **self- test has failed**



If the devices fails the test, a test summary provides details about the failure. Follow the instructions prior to performing the user test :

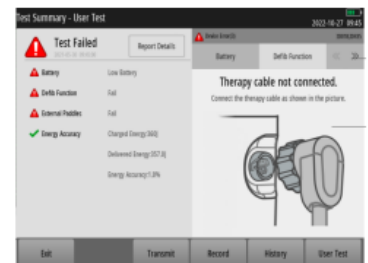

If the test has failed it is advised you run the test manually. To do this follow this procedure:

1. Ensure the device is connected to AC power.
2. Ensure all cables are clean and intact.
3. Press the power switch  to turn on the device.
4. Select the **Main Menu** key (there may be a delay while the D30 initialises fully).
5. Select **User test**.
6. Confirm you want user test mode and end patient monitoring by selecting **OK**.
7. Select **User test**, review the warning messages and select **Confirm** to continue.
8. When prompted push the **Charge** and **Shock** button.
9. Repeat as prompted to test battery power.
10. Perform operations following the displayed instructions, until the equipment prompts the test has been passed.

If the test fails again please report the device to your local Medical Engineering team.
To see previous tests press the  on screen option from the user test menu.

